Quick Start Guide

Ayre™ PRO
Wireless Rapid-Recovery
Compression Boots

Quick Start Guide

Ayre™ PRO Wireless Rapid-Recovery Compression Boots

Ayre™ Pro Boots

- A** 420D Nylon Boots
- B** Hook and Loop
- C** Zipper
- D** C1, C2, C3, C4 Chambers
- E** Charger socket

Ayre™ Pro Control Unit

- 1** Power On/Off and timer adjustment button
- 2** Air pressure button
- 2+1** Mode selection button

Intended use

The Compex Ayre™PRO is a battery-powered electric massager indicated to temporarily relieve tension and relax muscle following exercise and to temporarily increase circulation to the affected areas. Do not use Compex Ayre™PRO for the treatment of any illness or injury or on damaged skin or tissue.

This product is a non-medical device and is not used in medical conditions or medical environments.

Contraindications

This device must not be used for the following conditions:

- Persons with suspected, active or untreated: deep vein thrombosis, ischemic vascular disease, severe arteriosclerosis, pulmonary edema, severe congestive heart failure, thrombophlebitis or an active infection.
- On the legs where cuffs would interfere with the following conditions: vein ligation, gangrene, dermatitis, open wounds, a recent skin graft, massive edema or extreme deformity of the leg.
- Not for use on any neuropathy.
- Do not use on extremities that are insensitive to pain.
- Not for use where increased venous or lymphatic return is undesirable.

Warnings

- Do not attempt to repair the device.
- Do not attempt to open or remove covers.
- Do not remove the pump unit from the cuff.
- Do not attempt to modify or change the device.
- Never attempt any service while the device is in use.
- Do not operate in a wet environment.
- Do not immerse in any liquid for any reason.
- For cleaning and disinfecting instructions refer to Clean and Disinfecting section.
- Do not place the device in autoclave for any reason.
- Not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- If exposed to temperatures below 10° (50°F) allow the device to warm up to room temperature.
- Do not subject the device to extreme shocks, such as dropping the pump.
- Portable and mobile Radio Frequency Communication Equipment can be affected by this device.

Precautions

- Stop using the device if swelling, skin irritation or any other unpleasant or painful sensation occurs and consult a physician.
- Loosen cuffs immediately if pulsation or throbbing occurs as the cuffs may be wrapped too tightly.
- For people with diabetes or vascular disease that require frequent skin assessment, consult a physician.
- For people who use warming devices in combination with cuffs that require frequent assessment as skin irritation may occur; consult a physician.

Quick Start Instructions

Read the full user manual thoroughly (available at compex.com)

- Place your foot in the wrap, zip up the boot, and secure the scratch to hold your leg in place. Make sure the wrap is snug, but not too tight.
- Hold the On/Off button for 1 second to turn on the device.
- The device will begin to operate in the default mode F1 which will inflate each of the four chambers to the default pressure setting (50 mmHg). To change the pressure settings press the Set button to circle through the different level. After setting up the air pressure of one chamber, press and hold the Set button for 1 second to set up the pressure of the next chamber.
- To change the operating mode, press and hold the Power button and the Set button simultaneously for 2 seconds. Release both buttons once the mode has successfully switched.
- Hold the On/Off button for 1 second to turn off the device.

Charging the Control Unit

The battery icon will flash during charging and will hold when fully charged. The device may be used while it is charging.

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enovis[™]

Compex is a brand of Enovis.

Enovis[™] is a medical technology company. Enovis is strongly focused on developing clinically differentiated therapeutic solutions that generate measurably better patient outcomes and transform workflows. Powered by a culture of continuous improvement, extraordinary talent and innovation—we “create better together” by partnering with healthcare professionals. Together, we help patients thrive through an extensive range of products, services and integrated technologies that fuel an active lifestyle.

COPYRIGHT © 2025 by DJO, LLC

Individual results may vary. Neither DJO, LLC nor any of its subsidiaries dispense medical advice. The contents of this document do not constitute medical advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

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Copyright © 2025 DJO, LLC, a subsidiary of Enovis Corporation
DJO France S.A.S, Centre Européen de Frêt, 3 rue de Bethar, 64990 Mouguerre, FRANCE