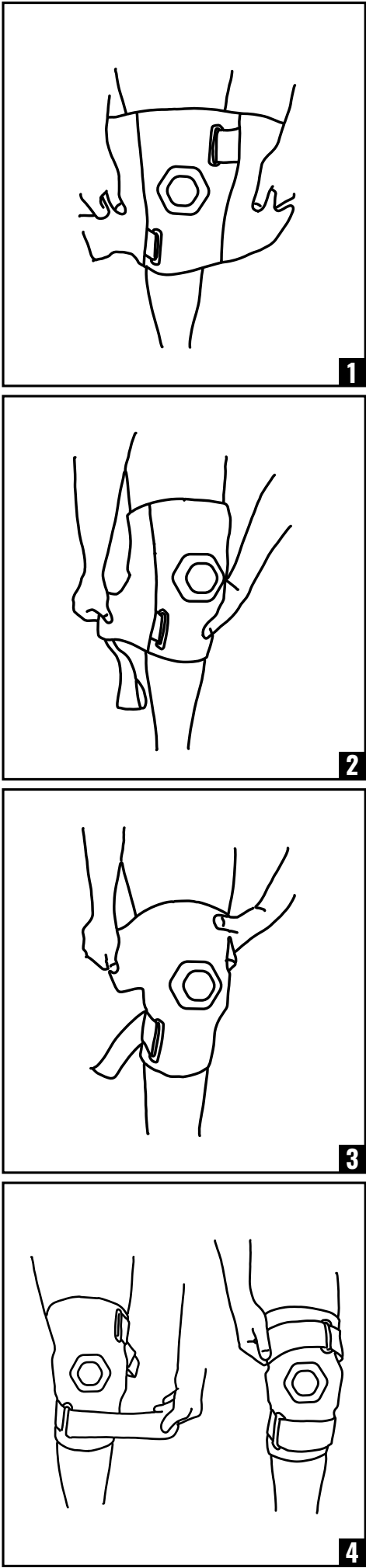
ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USE/INDICATIONS: Medial/Lateral Ligament Protection, Knee Instability, Sprains and Strains, Hyperextension, Meniscus Injuries

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

- Loosen straps.
- Center the brace and knee buttress over the center of the knee.
- Wrap the bottom flap behind the leg and secure, repeat for top flap.
- Feed the bottom strap through loop and attach, repeat for the top strap.

CLEANING INSTRUCTIONS: Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Neoprene 48%, Polyester 18%, Nylon 22%, Rubber 7%, Silicone 5%

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY. LATEX

Caution: This product contains natural rubber latex which may cause allergic reactions.

LEVEL OF SUPPORT

MIN MID MAX MAX+

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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