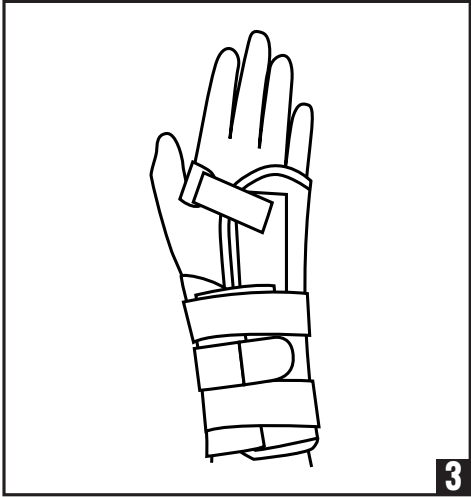
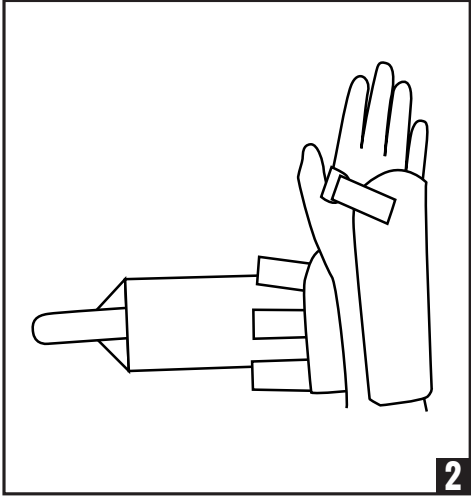
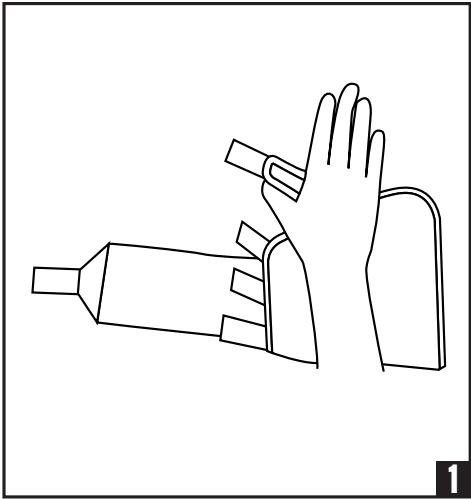
ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USE/INDICATIONS: Carpal Tunnel, Mild Sprains and Strains, Tendonitis, Wrist Instabilities

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Open all straps.
2. Place hand into brace with rigid support on the palm side.
3. Attach straps, starting with the hand, then the three Velcro straps and finishing with the circumferential strap. Adjust as needed.

CLEANING INSTRUCTIONS:

 Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Nylon 63%, Polyester 8%, PU 19%, Rubber 10%

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

Caution: This product contains natural rubber latex which may cause allergic reactions.

LEVEL OF SUPPORT

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2024 DJO, LLC