
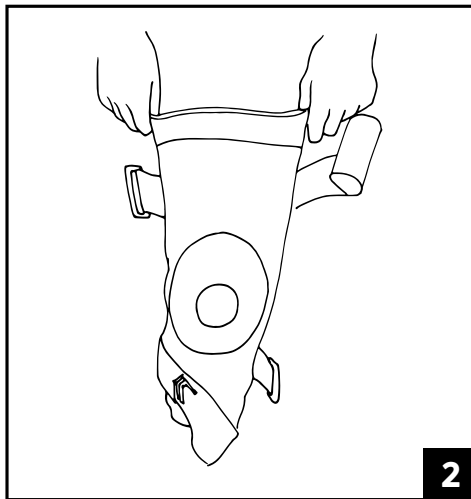
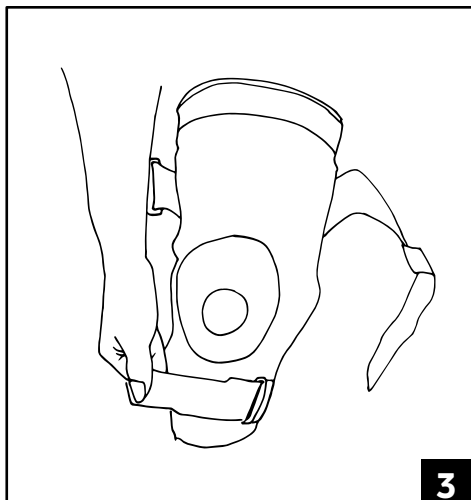
1

2

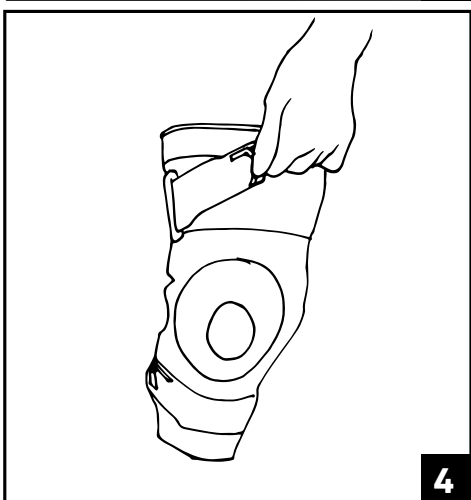3

4

ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

RECOMMENDED FOR: Sprains and Strains, Patellofemoral Syndrome, Inflammation/Swelling

CONTRAINDICATIONS: None

WARNINGS AND CAUTIONS: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Unfasten straps
2. Pull sleeve up onto leg and center the knee buttress over the center of knee cap
3. Feed the bottom strap through the D-ring and secure. Repeat for the top strap.

TO CONVERT TO LEFT KNEE BRACE:

1. Unfasten the top strap completely
2. Locate the Hinge Slits at the top of the hinge pockets
3. Holding the center of the hinge, pull on the top of the fabric to expose the hinge
4. Remove the hinges by sliding out of the opening and reinsert on opposite side.
5. Top of hinges are labeled according to the side they should be located. Hinge descriptions should face outward.
6. Hinge description on top should now read Outside Left Knee and Inside Left Knee.

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Nylon 54%, TPE 19%, Spandex 11%, Polyester 8%, Neoprene 8%

WARRANTY: DJO, LLC WILL REPAIR OR REPLACE ALL OR PART OF THE UNIT AND ITS ACCESSORIES FOR MATERIAL OR WORKMANSHIP DEFECTS FOR A PERIOD OF SIX MONTHS FROM THE DATE OF SALE.

INTENDED FOR SINGLE PATIENT USE.

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2022 DJO, LLC