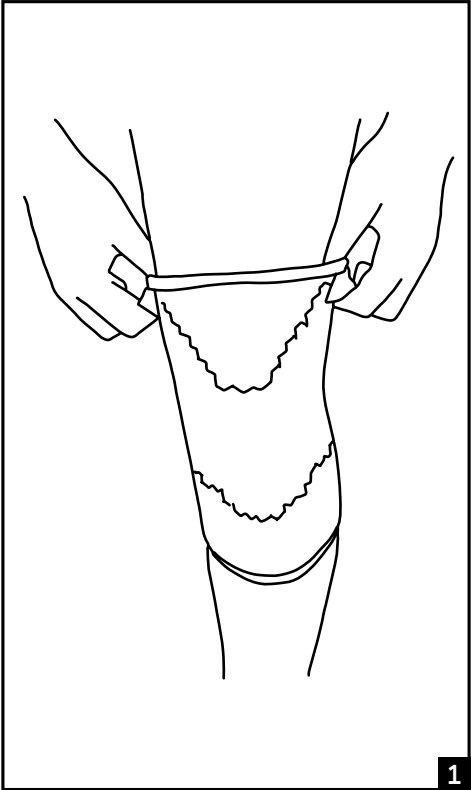
1

ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

RECOMMENDED FOR: Knee Support, Compression, Stabilization, General Knee Pain, Swelling, Mild Strains and Sprains

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

- Step through sleeve/s
- Use pull tabs to pull sleeve up until hex pattern is centered over the knee

CLEANING INSTRUCTIONS:

 Hand wash in warm water using mild soap, rinse thoroughly

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

63% Neoprene, 31% Polyester, 2% Nylon, 1% TPU, 1% Spandex, 2% Rubber

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

INTENDED FOR SINGLE PATIENT USE.

 **PRECAUTION:** This product contains natural rubber latex which may cause allergic reactions

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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