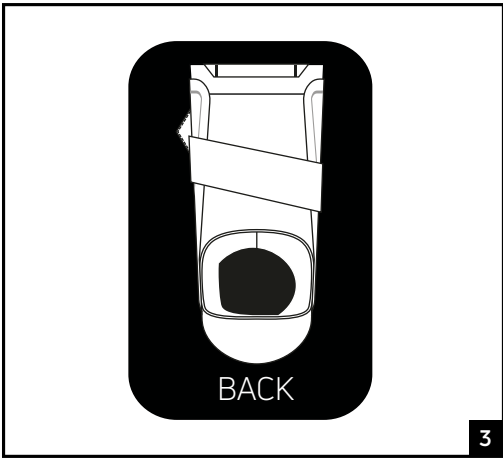
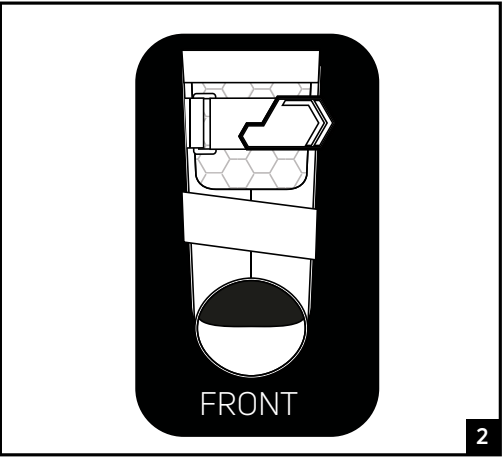
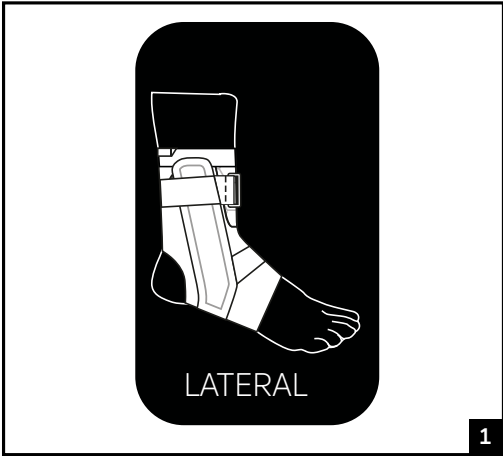
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

RECOMMENDED FOR: Mild to Moderate Ligament Sprains, Mild to Moderate Tendon Strains, Ankle Instability.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products.
- Do not use over open wounds.
- Do not use the device if it is damaged.
- The support should be snug but not impair circulation.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1. Pull brace on until stabilizer is comfortably secured in the plantar fascia (arch). (Fig. 1)
2. Wrap stretch webbing strap around the back of the foot (above the medial malleolus) and thread through the D ring tensioner. (Fig. 2)
3. Tighten brace to desired compression. (Fig. 3)

CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Main Body: Polyester, Neoprene

Straps: Nylon, Elastane

Pull tabs: Silicone

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

FOR SINGLE PATIENT USE ONLY.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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