ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INDICATIONS: The DonJoy Bionic Back Wrap is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain.

PERFORMANCE CHARACTERISTICS: Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS: If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

- WARNINGS AND PRECAUTIONS:**
- For maximum efficacy, ensure the correct size of strap is selected.
 - If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.
 - Do not use over open wounds.
 - Do not use this device if it was damaged and/or package has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

1. Loosen the two external straps and open up back wrap so it can lay flat
2. Slide fingers into the designated finger slots (Figure 1)
3. Wrap torso and affix hook and loop (Figure 2)
4. Tighten external straps to desired compression

CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Main Body: Polyester, EVA

Straps: Polyamide, Elastane

Pull Tabs: Silicone

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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