
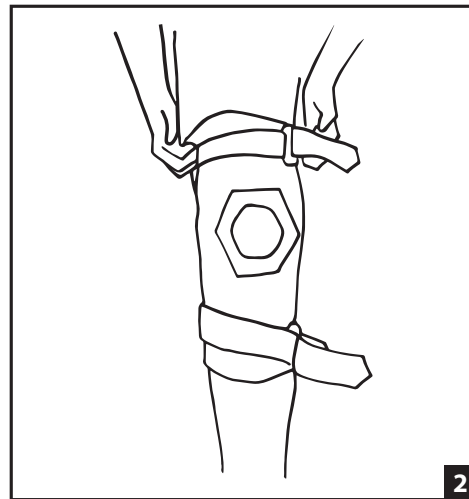
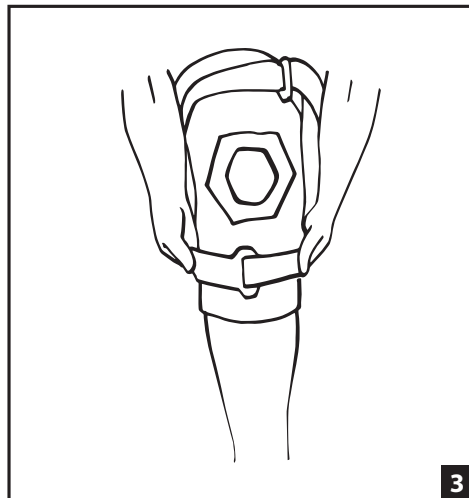
1

2

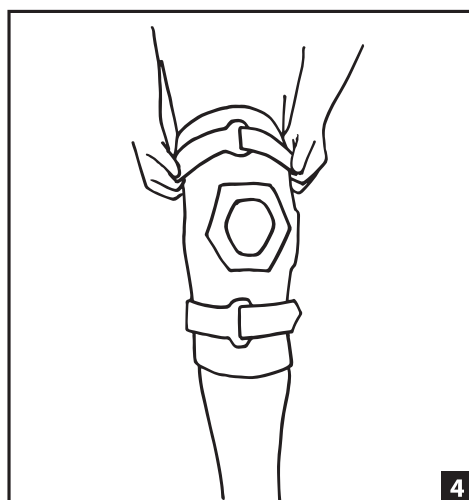3

4

ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

RECOMMENDED FOR: The DonJoy Bionic Drytex Knee Sleeve is designed to support the knee. It may be suitable for use following knee sprains, strains and soft-tissue injuries and may help those suffering from arthritis and instability. Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The support should be snug but not impair circulation.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

CAUTION: This product contains natural rubber latex which may cause allergic reactions

APPLICATION INFORMATION:

1. Loosen straps. (Fig. 1)
2. Use pull tabs, step into the sleeve. (Fig. 2)
3. Pull sleeve up until the hex buttress is centered over the knee cap. (Fig. 3)
4. Feed the bottom strap through the D-rings and secure. Repeat for top strap. (Fig. 4)

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Polyester 54.7%, Nylon 18.7%, Silicone 9.6%, Spandex 6.7%, Neoprene 5%, EVA 2.3%, TPU 2%, Rubber 1%

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

INTENDED FOR SINGLE PATIENT USE.**LEVEL OF SUPPORT**

MIN MID MAX MAX+

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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