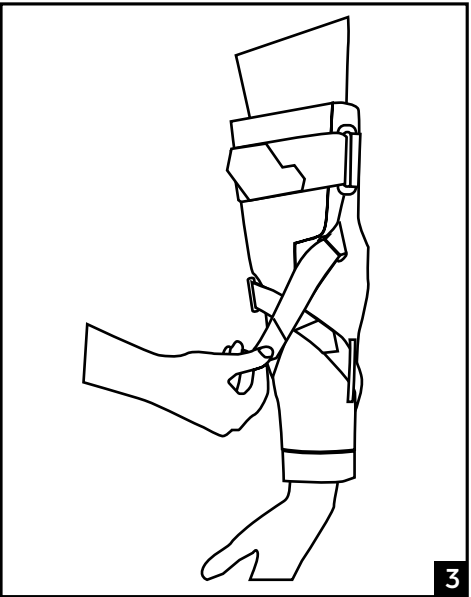
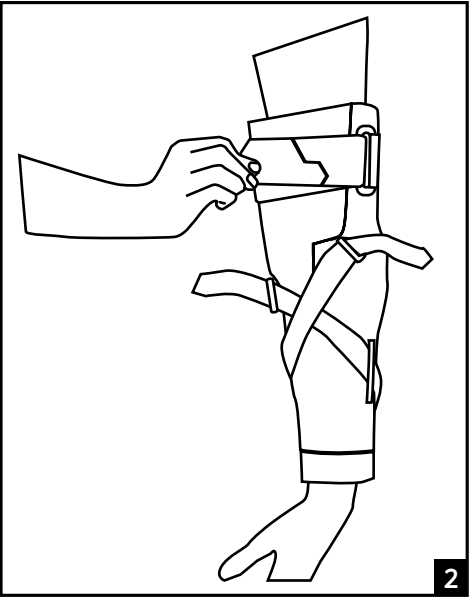
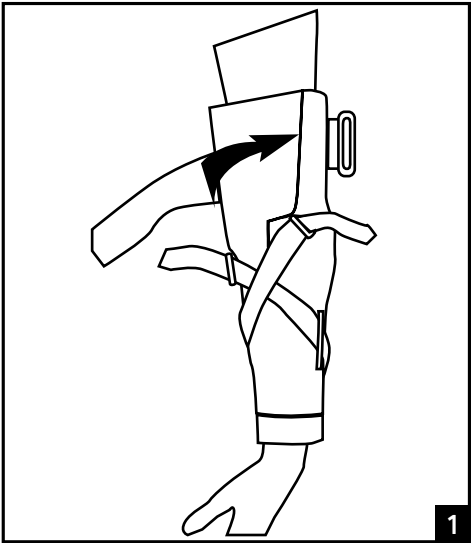
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

RECOMMENDED FOR: Protection from Medial/Lateral Forces, Joint Instability, Mild to Moderate Sprains and Strains, Hyperextension.

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Loosen all straps and undo hook and loop closure at the top.
2. Slide the brace on the arm until the hinges are sitting one finger above the elbow joint.
3. Affix the hook to the loop around the bicep.
4. Thread the stretch webbing bicep strap through the D ring tensioner and tighten to desired compression.
5. Tighten the cross strapping straps across forearm to desired angle of extension.

CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Main Body: Neoprene, Polyester,

Straps: Nylon;

Pull-Tabs: Silicone;

Hinge: Aluminum

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

INTENDED FOR SINGLE PATIENT USE.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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