
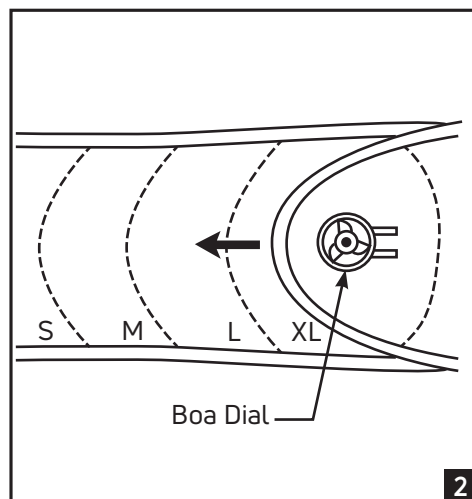
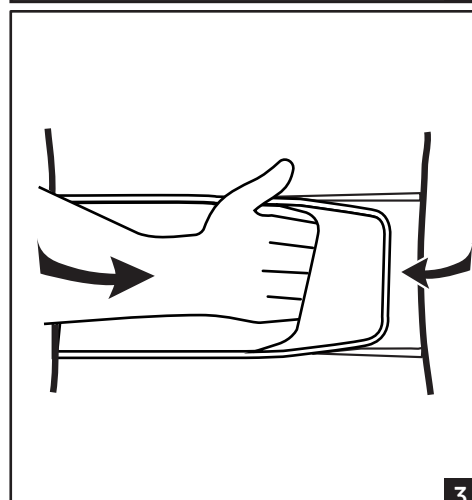
1

2

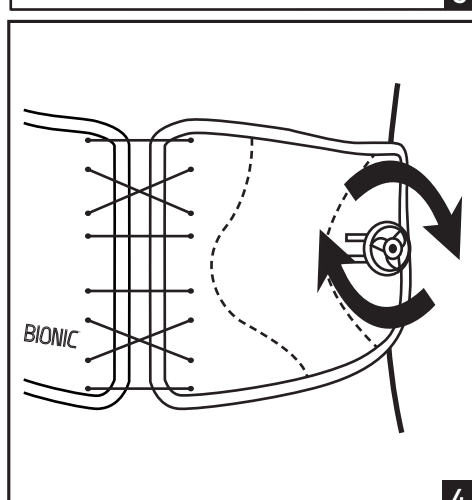3

4

**ENGLISH**

**BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.**

**RECOMMENDED FOR:** Low back pain, Lumbar Control, Low Back Sprains, and Strains

**CONTRAINDICATIONS:** Pregnancy and circulation, pulmonary, cardiovascular or skeletal conditions which have risk to be made worse as a result of compression and/or pressure.

**WARNINGS AND CAUTIONS:** Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

**APPLICATION INFORMATION:**

1. Open up the brace and lay with Boa® knobs and silver panel face-up. Lift the Boa® knobs to loosen the lacing system. See figure 1.
2. Adjust the brace size by lifting each of the silver panels and aligning with one of the lime green semi-circles on each side. See figure 2 for sizing guide.
3. Align lacing system over your spine, keeping "BIONIC" logo legible. Slide your right hand into the finger slots to wrap brace around waist. Secure the contact closure, right over left, so the brace is snug. See figure 3.
4. Push the Boa® knobs down and turn for micro-adjustment and additional compression. See Figure 4.

**CLEANING INSTRUCTIONS:**

Hand wash in warm water using mild soap, rinse thoroughly

**Note:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**MATERIAL CONTENTS:**

51% POLYESTER, 25% NYLON, 11% NEOPRENE, 10% EVA, 3% SPANDEX

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**INTENDED FOR SINGLE PATIENT USE.**

**NOT MADE WITH NATURAL RUBBER LATEX.**

**LEVEL OF SUPPORT:**

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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