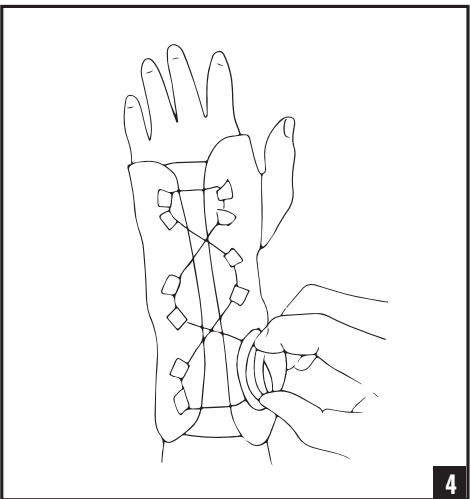
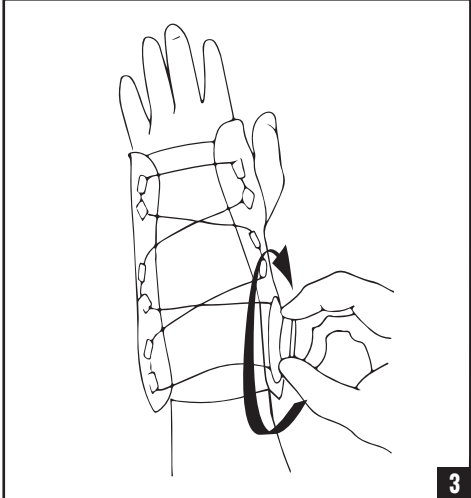
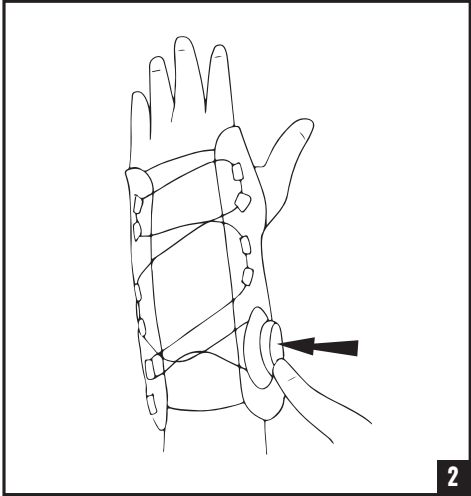
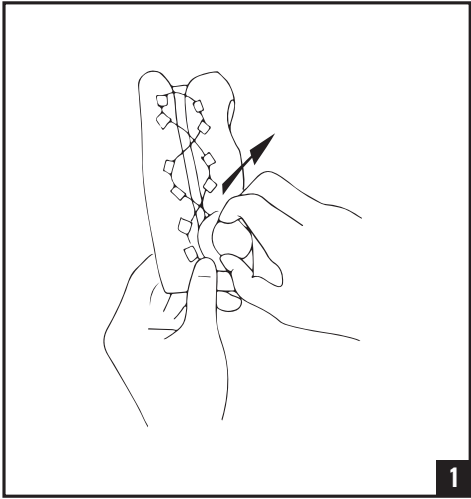
ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

RECOMMENDED FOR: Carpal Tunnel, Mild Sprains and Strains, Tendonitis, Wrist Instabilities

CONTRAINDICATIONS: None

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

- Loosen brace by lifting up on the BOA dial until it clicks
- Slide hand into the sleeve until the thumb is positioned in the thumb opening
- Push dial down until it clicks and turn the dial clockwise to desired fit
- To loosen the brace, lift the BOA dial up until it clicks than re-tighten by turning the dial clockwise

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY.
Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

75% Nylon, 10% Polyester, 10% Spandex, 3% Polyurethane, 2% Polypropylene

WARRANTY: DJO, LLC WILL REPAIR OR REPLACE ALL OR PART OF THE UNIT AND ITS ACCESSORIES FOR MATERIAL OR WORKMANSHIP DEFECTS FOR A PERIOD OF SIX MONTHS FROM THE DATE OF SALE.

INTENDED FOR SINGLE PATIENT USE.

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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DJO, LLC
5919 Sea Otter Place
Suite 200
Carlsbad CA 92010 USA

EC REP

UK Responsible Person
MDSS-UK RP LIMITED
6 Wilmslow Road, Rusholme,
M14 5TP Manchester,
United Kingdom

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©2024 DJO, LLC