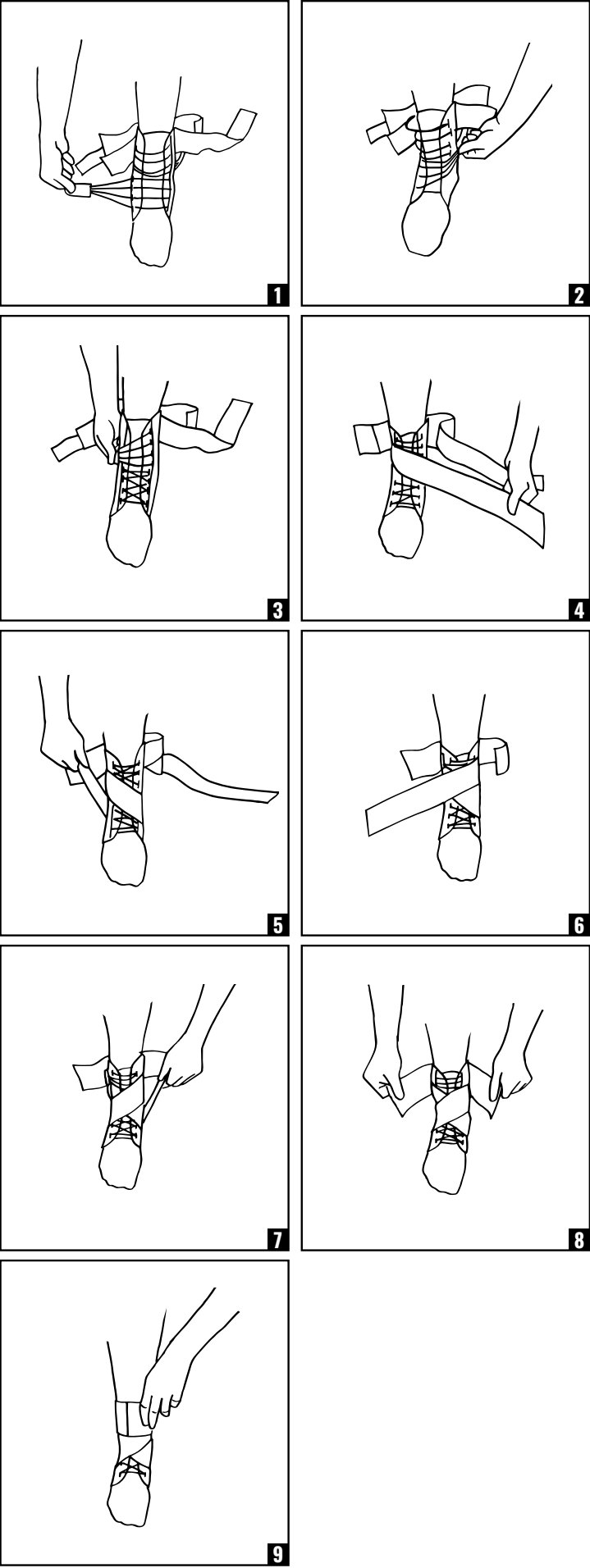
ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USE/INDICATIONS: Medial/Lateral Ligament Protection, Ankle Sprains, Ankle Instability

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Apply sock, loosen straps and laces.
2. Slide foot into brace toes first, ensure the tongue is on top of the foot.
3. Pull bottom speed-lace strap tab across the top of foot and secure on the opposite side of ankle.
4. Pull top speed-lace strap tab across the top of foot and secure on the opposite side of ankle.
5. Take the outer strap and wrap over forefoot, under arch, pull and attach strap up the outside of the ankle to desired tension.
6. Take the inner strap and wrap over forefoot, under arch, pull and attach strap up the inside of the ankle to desired tension. Straps should now form a "X" over top of foot.
7. Pull the top circumfrence straps towards front of ankle over laces and secure.
8. Remove medial/lateral stays as needed for custom level of support.

CLEANING INSTRUCTIONS:

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Polyester 46%, Nylon 32%, Silicone 12%, Rubber 5%, Foam 4%, Cotton 1%

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

FOR SINGLE PATIENT USE ONLY.

LATEX

Caution: This product contains natural rubber latex which may cause allergic reactions.

LEVEL OF SUPPORT

MIN

MID

MAX

MAX+

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2021 DJO, LLC