ENGLISH

BEFORE APPLYING THE BRACE, PLEASE READ THESE INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE BRACE.

INTENDED USE/INDICATIONS: Patellofemoral Syndrome, Mild Sprains and Strains, Knee Instabilities, Inflammation/Swelling, Pain Relief, Soreness, Arthritis, Bursitis, Overuse Injuries

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

- Slide sleeve into place with the knee cap located within the patella buttress and the logo centered over the knee. The stays should fall on the inside and outside of the knee.

CLEANING INSTRUCTIONS:

 Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Nylon 79%, Spandex 10%, PU 2%, Polyester 9%

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

INTENDED FOR SINGLE PATIENT USE.

 NOT MADE WITH NATURAL RUBBER LATEX.

LEVEL OF SUPPORT

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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