ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

RECOMMENDED FOR: The DonJoy Performance Bionic FullStop ACL Brace is designed to support the knee and provide protected range of motion following ACL injuries, moderate ligament sprains and meniscus injuries. It may also benefit those suffering from medial/lateral ligament instability and joint instability by providing patella support/stability. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing mild protection to a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. • Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure. • ROM settings should not be changed without supervision of a medical professional. • This product must be prescribed and fitted by a healthcare professional. • The frequency, duration of use and directions for use should be determined by your healthcare professional. • Do not use over open wounds. • Do not use the device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INFORMATION:

- Loosen all four straps.
- Pull the brace on and center the patella in the hex shaped patella opening.
- Ensure the brace is aligned with both hinges running symmetrically down the side of the leg.
- Tighten the top strap to desired tightness.
- Tighten the bottom strap to desired tightness.
- Tighten the middle, lower strap to desired tightness.
- Tighten the middle, upper strap to desired tightness. (This strap does not wrap the entire leg, it only crosses the back of leg)
- Once all four straps have been tightened, make any micro adjustments needed for brace to fit comfortably.

CLEANING INSTRUCTIONS:

Hand wash in water (86°F/30°C) using soap. Air dry.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Main Body: Polyester, Neoprene, Elastane;

Pull Tabs: Silicone, Polyurethane;

Hinge: Aluminum

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

RX ONLY.

INTENDED FOR SINGLE PATIENT USE.

NOT MADE WITH NATURAL RUBBER LATEX.

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2024 DJO, LLC