
1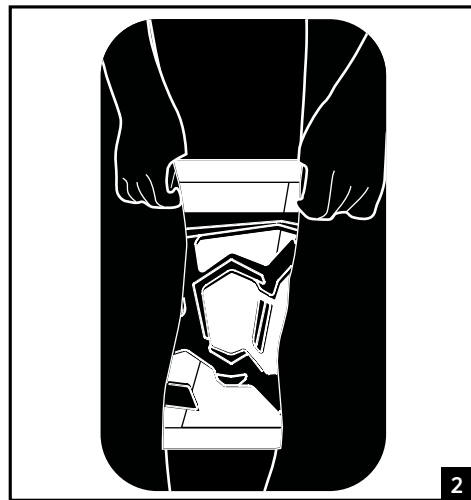

2

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**RECOMMENDED FOR:** Strains, Soreness, Repetitive Motion Injuries, Proprioception, Inflammation/Swelling, Degenerative Meniscus, Tendonitis, Mild Joint Instability, Compression.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained in this product.

**INTENDED USE/INDICATIONS:** The DonJoy 2MM Knee Sleeve is designed to support the knee. It may be suitable for use following strains, sprains and soft tissue injuries and, may help to enhance patella stability. Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

### WARNINGS AND PRECAUTIONS:

- The support should be snug but not impair circulation.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or package has been opened.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

1. Pull sleeve on until the patella is in the center of the silicone opening. (Island silicone piece always sits on the medial side of the patella). (Figure 2)

### CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

**Note:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

### MATERIAL CONTENTS:

**Main Body:** Polyester, Carbonized Bamboo;

**Banding:** Silicone

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### FOR SINGLE PATIENT USE ONLY.

 NOT MADE WITH NATURAL RUBBER LATEX.

### LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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