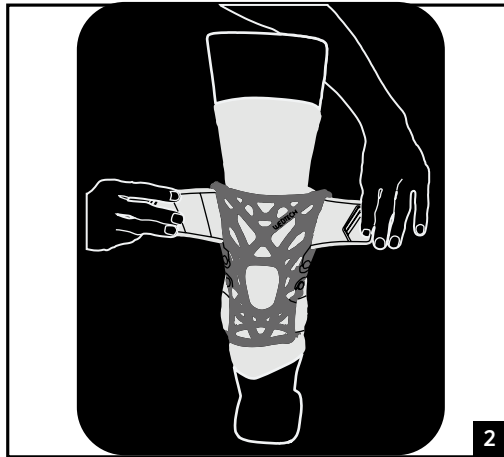
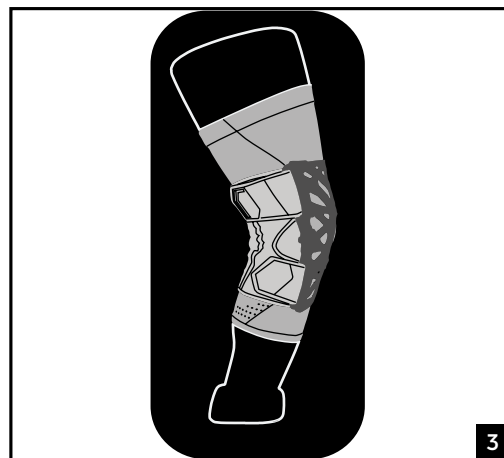
1

2

3

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**RECOMMENDED FOR:** The DonJoy WebTech Knee is designed to support the knee. It may be suitable for those suffering from knee pain, Degenerative Conditions such as osteoarthritis, Tendonitis, Chronic Tendinosis, Chondromalacia, Osgood Schlatters, and Patellofemoral Tracking.

**PERFORMANCE CHARACTERISTICS:** Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

**CONTRAINDICATIONS:** Do not use if you are allergic to any of the materials contained in this product.

### WARNINGS AND PRECAUTIONS:

- Do not use over open wounds.
- Do not use this device if it was damaged and/or package has been opened.
- The support should be snug but not impair circulation.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INFORMATION:

1. Pull on undersleeve until comfortable and patella is in the center of the parallel stitch lines. Wearing undersleeve is optional. (Figure 1)
2. Lay web over the top of the knee joint, centering the web by aligning the opening with the patella. Outside of web has D-Hex logo. (Figure 2)
3. Thread two tabs through the D-ring openings and affix hook portions to the loop of the backpad.
4. Repeat for top tabs. Brace should be snug and comfortable, do not over tighten. (Figure 2)
5. Once all tabs are affixed, to ensure proper tightness, lift and pull both lower tabs until snug. (Figure 3)

### CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

**Note:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

### MATERIAL CONTENTS:

**Main Body:** Polyester, Elastane;

**Backpad:** Nylon, Polyurethane;

**Web:** Silicone, Nylon

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

### FOR SINGLE PATIENT USE ONLY.

NOT MADE WITH NATURAL RUBBER LATEX.

### LEVEL OF SUPPORT:

MIN MID MAX MAX+

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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