Figure 1

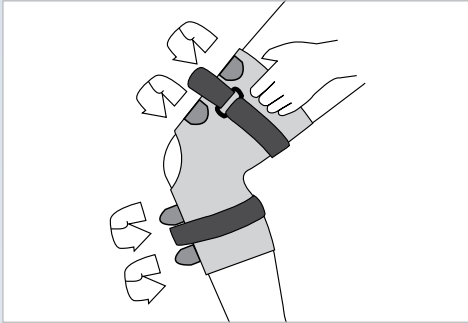
Figure 2

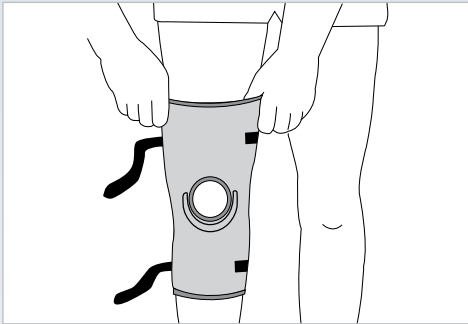
Figure A

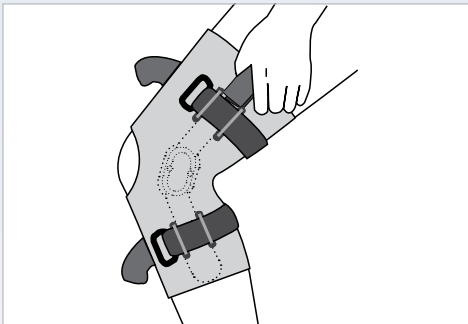
Figure B

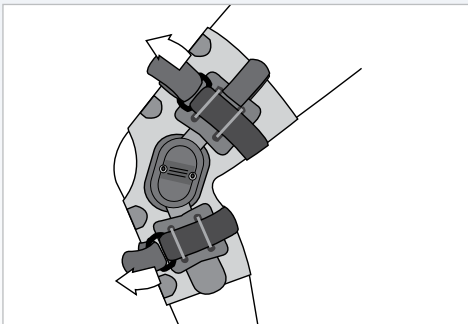
Figure C

## ENGLISH

**BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.**

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

The DonJoy Performer Patella Knee Brace is designed to support the knee. It may be suitable for those suffering from mild knee instabilities, sprains, strains, tendinitis, and arthritis.

Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

### CONTRAINDICATIONS

- Phlebitis patients should not wear the Performer Knee Support.

### WARNINGS AND PRECAUTIONS

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
  - Do not use over open wounds.
  - Do not use this device if it was damaged and/or package has been opened.
  - The support should be snug but not impair circulation.
  - If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.
- NOTE** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device

### APPLICATION INSTRUCTIONS

#### Patella Stabilizer:

- Grasp the edges of the stabilizer at the larger opening. Step through the stabilizer, bringing it onto the knee so that the opening at the front surrounds the kneecap. (Fig. 1)
- Fasten the support straps at the thigh and calf. (Fig. 2)

#### Hinged Patella Stabilizer:

- Grasp the edges of the stabilizer at the larger opening. Step through the stabilizer, bringing it onto the knee so that the opening at the front surrounds the kneecap. (Fig. A)
- Position the sleeve so that the hinges are parallel to each other, and the top of the circular cutout aligns with the top of the kneecap. (Fig. B)
- Fasten the support straps at the thigh and calf. (Fig. C)

### CARE

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritating and deteriorate material. The brace may be used in fresh or salt water. Following use, drain thoroughly and rinse with clean, fresh water. AIR DRY.

NOT MADE WITH NATURAL RUBBER LATEX.

### FOR SINGLE PATIENT USE ONLY

**WARRANTY** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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©2023 DJO, LLC
MDSS CH GmbH
MDSS GmbH