ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

RECOMMENDED FOR: Inversion and Eversion Ankle Sprains, Ankle Instability.

CONTRAINDICATIONS: N/A

WARNINGS AND CAUTIONS: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene byproducts. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. The support should be snug but not impair circulation. Never wear the support over an open wound.

APPLICATION INFORMATION:

1. Loosen or remove the ankle strap.
2. Peel the hook and loop TPR pull tabs and open the ladder ratchet closure system and lining so that the front of brace is wide open.
3. Step into brace and snugly close the lining around ankle and affix hook patch to loop. (Note, heel should protrude past the heel cup on the footplate)
4. Feed the lateral TPR pull tab through the slotted opening in the medial strap and pull the TPR pull tabs until desired tightness.
5. Footplate should fit comfortably in the arch of foot.

CLEANING INSTRUCTIONS:

 Hand wash in water (86°F/30°C) using soap. Air dry.

Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

MATERIAL CONTENTS:

Cuff Pad: Polyester, EVA;

Cuff: Zytel;

Footplate: Nylon/ABS

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

INTENDED FOR SINGLE PATIENT USE.

 **NOT MADE WITH NATURAL RUBBER LATEX.**

LEVEL OF SUPPORT:

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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